SLENDERTONE®

CoreFit

Instruction Manual
Instrucciones de Uso
Mode D'emploi

www.slendertone.com

WELCOME

Congratulations on purchasing SLENDERTONE COREFIT to begin your journey to getting firmer and flatter abs. Please read the user manual fully before using this product.

If you have any difficulty setting up your product or have any other questions or concerns, please send us an email or call the SLENDERTONE Careline on:

- Tel: 1-800-551-2443
- Email: info@slendertoneusa.com
- Web: www.slendertone.com

CONTRA-INDICATIONS, WARNINGS & PRECAUTIONS

Your SLENDERTONE product is indicated for the improvement of abdominal muscle tone, for strengthening of abdominal muscles, and for the development of a firmer abdomen.

Your SLENDERTONE product is suitable for use by all healthy adults; however, as with other forms of exercise, some care is needed when using it. Always follow the guidelines below and read the instruction manual before use. Some of the points below are gender-specific.

Please do not use if:

- You have an electronic implant (e.g. cardiac pacemaker or defibrillator - as your SLENDERTONE product may interfere with the proper functioning of the implanted stimulator) or suffer from any other heart problem.
- You have any other form of abdominal implant. The safety has not been established for the use of electronic muscle stimulators over abdominal implants.
- You are pregnant. The safety of electronic muscle stimulation (EMS) over the pregnant uterus has not been established.
- You suffer from cancer. The effects of electronic stimulation on cancerous tissue are unknown.
- You are under medical supervision for cognitive dysfunction as you may not be able to comply with safety instructions.
- The controller is in close proximity (e.g., 1 yard) to shortwave or microwave therapy equipment or you are connected to high-frequency surgical equipment (e.g. diathermy), because of the risk of device malfunction due to electronic interference.
- Wearing the device necessitates placement over areas at which drugs/medicines are administered (short-term or long-term) by injection (e.g., hormone treatments).

WARNINGS

- The long-term effects of chronic electronic stimulation are unknown.
- Product should not be used while sleeping, because you cannot react to safety information.

Please wait before using your SLENDERTONE product until:

- At least 6 weeks after the birth of your baby. You must consult your doctor before use.
- One month after an IUD contraceptive device (e.g., coil) has been fitted. You must consult your doctor before use.
- At least 3 months after having a Caesarean section. You must consult your doctor before use.
- The heavy days of your period have finished because vigorous abdominal exercise is not recommended at this time.

When applying the GelPads™ and belt, always remember to:

- Place the GelPads and belt ONLY on the abdomen as indicated in this manual.
- Avoid placing the GelPads or belt on the front or sides of the neck, across or through the heart (i.e. one GelPad on the front of the chest and one on the back), in the genital region or on the head, because of the risk of stimulating inappropriate muscles and organs.
- Avoid application of GelPads near the thorax as this may increase the risk of cardiac fibrillation.
- Avoid any recent scars, broken or inflamed skin, areas of infection or susceptibility to acne, thrombosis or other vascular problems (e.g. varicose veins), or any parts of the body where feeling is limited.
- Avoid areas of injury or restricted movement (e.g. fractures or sprains).
- Avoid placing the GelPads over metal implants. They may be placed on the nearest muscles.
- Do not switch the controller on until it has been correctly inserted into the holster on the belt.

PRECAUTIONS:

- Caution should be used for patients with suspected or diagnosed epilepsy.
- Caution should be used in the following:
 - When there is a tendency to bleed internally following an injury or fracture.
 - Following recent surgical procedures when electronic stimulation may disrupt the healing process.
 - Over areas of skin which lack normal sensation.
- Your SLENDERTONE product is not intended for medical use, for the treatment of any medical condition, nor for any permanent physical changes.
- Always allow two hours for the controller to cool down between stimulation sessions. During normal use the controller generates a small amount of heat and tests have shown that in extreme conditions this can cause a temperature rise up to 2°C / 3.6°F under the belt at the controller location. At high ambient temperatures, close to 40°C / 104°F, care is needed to avoid prolonged exposure to this increased temperature.

Get your doctor's permission before using your SLENDERTONE product if:

- You wish to use this product and have a bad back. You should also ensure the intensity is kept low.
- You have any serious illness or injury not mentioned in this guide.
- You have recently undergone a surgical procedure.
- You take insulin for diabetes.
- You suffer from muscle or joint problems.

Possible adverse reactions:

- Skin irritations and burns have been reported with the use of powered muscle stimulators.
- On very rare occasions, first-time users of EMS have reported feeling light-headed or faint. We recommend that you use the product while seated until you become accustomed to the sensation.

To reposition GelPads during a session:

- Always pause the program currently running, unfasten the belt and then refasten it behind your back after the GelPads' positions have been adjusted.

After strenuous exercise or exertion:

- Always use a lower intensity to avoid muscle fatigue.

Contact SLENDERTONE or an authorized dealer if:

- Your product is not working correctly. Do not use in the meantime.
- You experience any irritation, skin reaction, hypersensitivity, or other adverse reaction. You should, however, note that some reddening of the skin under the belt is quite normal during and for a short time after an exercise session.

NOTE:

- An effective session should not cause undue discomfort.

IMPORTANT:

- For first time users, muscle stimulation can be an unusual sensation. We recommend that you begin in a seated position with low stimulation intensity settings to familiarize yourself with the sensation before progressing to higher intensity settings.
- Keep your product out of the reach of children.
- The studs and GelPads must not be connected to other objects.
- Do not use your product at the same time as any other device which transfers an electrical current into the body (e.g., another muscle stimulator).
- Do not touch the GelPads or metal studs while the controller is switched on.
- Do not use while driving, operating machinery or cycling, or during any activity in which involuntary muscle contractions may put the user at undue risk of injury.
- The belt is for single person use only.
- Cease using your product if you are feeling light headed or faint. Consult your doctor if this happens.
- Do not over exert yourself when using muscle stimulation. Any workout should be at a level comfortable for you.
- Do not use your product if you are wearing a belly-button ring. Remove the ring before you begin a session.
- SLENDERTONE will not accept responsibility if the guidelines and instructions supplied with this product are not followed.
- Although compliant with applicable EMC requirements, this device may still interfere with more sensitive equipment, please move away or switch off.
- Keep this device out of the reach of pets or pests.

If you are in any doubt about using this product for any reason, please consult your doctor before use.

INTRODUCING YOUR SLENDERTONE COREFIT

PACK CONTENTS

1. Controller
2. Belt: Fits waist sizes 24" - 42"/61-106cm
3. Pack of three adhesive GelPads
4. USB Charging Cable
5. Instruction Literature

HOW THE CONTROLLER WORKS

1. On / Off / Pause Button (⏻)

Press this button to turn your controller on or off. You may also pause the exercise by briefly pressing this button during a toning session.

2. Intensity Up Button (+)

Press this button to increase the intensity of the exercise.


3. Intensity Down Button (—)

Press this button to decrease the intensity of the exercise.


NOTE: To increase or decrease intensity quicker, press and hold either intensity up/down button.

4. Program Button (P)

Press this button to select the toning program you wish to use. There are 10 programs in total, which are explained later in this manual (page 9).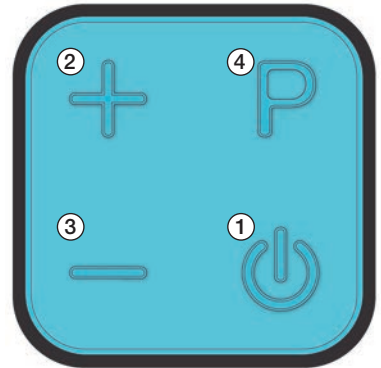

CONTROLLER DISPLAY (ICONS)

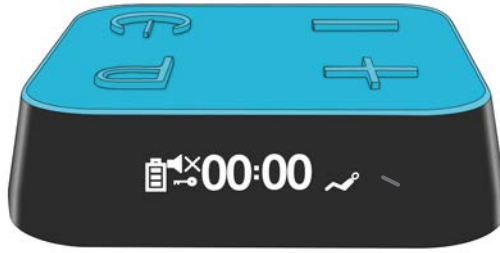The following are the icons which can appear on the screen during the treatment session.

| | |
|-----------------|---|
| P 01 | Program number |
| 49 | Toning intensity level |
| 23:49 | Countdown timer |
| | Session is paused |
| 🔊× | Sound is muted |
| 🔒 | Keylock is activated. You cannot change intensity levels or program whilst enabled. |
| ⏪ ⚠️ ⏩ | Load Sense Warning. Indicates a poor contact between the controller and belt or between the GelPads and skin (see page 13 for more details) |
| 🔋 | Battery charging level |
| ▲ 67 | Intensity increasing |
| ▼ 32 | Intensity decreasing |
| 56 ((∞)) | This indicates each contraction/relaxation phase - Programs 1-7 |
| 49 🧘 | Activity indicator for cardio activity - Program 8 |
| 64 🏃 | Activity indicator for active abdominal workout - Program 9 |
| 53' 🏃 | Activity arrow for the left side abdominal crunches - Program 10 |
| 57' 🏃 | Activity arrow for the right side abdominal crunches - Program 10 |
| ⚠️ 001 | Error message. Please see troubleshooting (page 13). |

When you switch on your controller, the screen will cycle through the following start-up screens. These indicate 1. Highest intensity reached in the previous session; 2. Number of sessions completed in current program; 3. Total number of sessions completed. You may skip these by pressing the Program button.

↑ 63 → #03 25 → # 50

SETTING UP YOUR SLENDERTONE COREFIT

Step 1 - Charge the battery.

We recommend plugging your charging cable into a USB port on your computer or other low-energy device. If necessary, only use a certified wall charger (e.g. UL/ETL) - Fig. a. Only use the charging cable that came with your device. It'll take up to three hours to fully charge the battery. Your controller is fully charged when you hear a beep and the battery icon on the display is flashing with all three bars full.

Fig. a


NOTES: Do not leave your controller connected to the charger when the battery is fully charged.

If the controller feels warm to the touch after charging, it should be allowed to cool down before use.


Step 2 – Place the GelPads on the belt.

- i) Remove the three adhesive GelPads from their pack. There is a large, square GelPad and two smaller oval GelPads. One side of each GelPad has a *grid pattern* while the other side is *black*.
- ii) Position the belt on a flat surface with the three metal studs facing upwards. Remove the plastic cover from the *grid pattern side* of the square GelPad and place the GelPad *grid pattern side* down within the blue square to cover the central metal stud. Press the edges of the GelPad firmly onto the belt.
- (iii) Then remove the plastic covers from the *grid pattern side* of the two oval GelPads and place them *grid pattern side* down either side of the square GelPad (Fig. b) in the positions best suited to your waist size. Press the edges of both GelPads firmly onto the belt.

Fig. b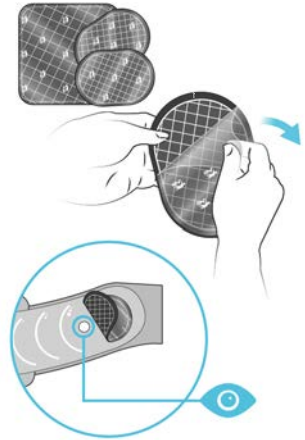

NOTE: The location of the two smaller GelPads depends on your waist size. In the example below (Fig.c) we show the smaller GelPads placed farthest from the large, square GelPad. This signifies the position for those who have a larger waist (36"-42" / 91-107cm). Use the sizing guidelines below to help you choose which is suitable for you.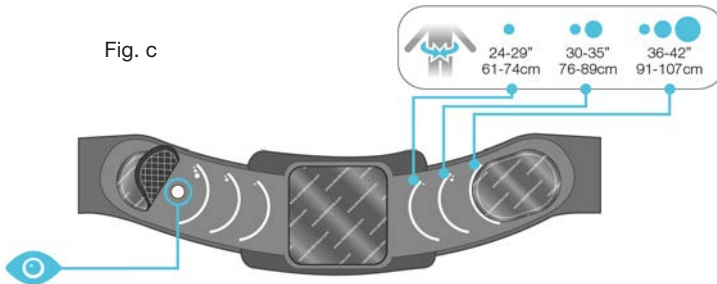


IMPORTANT: Ensure each GelPad is placed *grid pattern side down* and completely covering the metal studs. Press the edges of all three pads firmly onto the belt before use.

NOTE: The GelPads should be replaced every 25-30 sessions

Step 3 – Attach controller and remove the covers.

- i) Attach the controller to the belt ensuring you align the magnetic connectors on the rear of the controller to those on the holster, as shown in Fig. d.

Fig. d

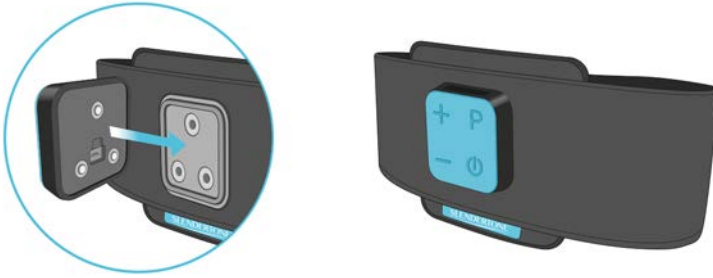- ii) Remove the plastic covers from the **black side** of all three GelPads as shown in Fig. e. Do not throw these plastic covers away as you will need to put them back onto the GelPads at the end of the session.


Fig. e

Step 4 - Position the belt on your body.

Wrap the belt around your waist so that the GelPads are pressed firmly against your skin, positioning the large square GelPad over your navel (belly button). Stretch the ends of the belt around your waist until the small GelPads are between your hip bone and ribs. Fasten the belt tightly around your waist, but not so tight as to cause discomfort. See Fig. f.

Fig. f

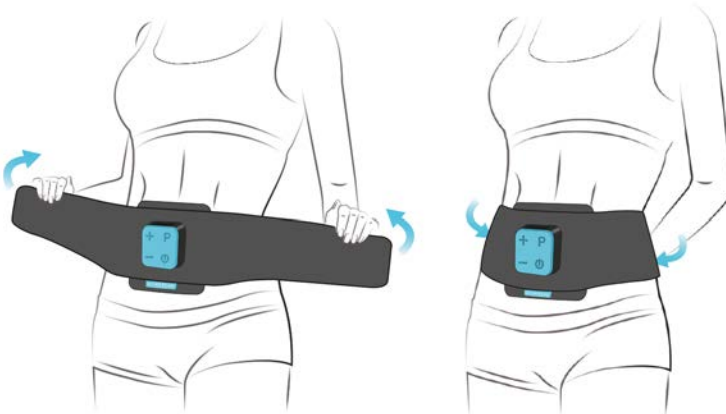NOTE: You can adjust the belt for a smaller waist size by folding over the left end and securing it with the additional Velcro tab. Then securely fasten the belt around your waist as normal.

YOUR FIRST TONING SESSION

Muscle toning can be an unusual sensation, but a pleasant one. We recommend that you use the product while seated until you become accustomed to the sensation. To get started, follow these simple steps:

Step 5:

- i) Press and hold the on/off button briefly to switch your controller on, which will illuminate the display screen and is indicated by a rising tone (see Fig. g). The controller will cycle through its three start-up screens and then display the current program.
- ii) Select the program you wish to use by pressing the Program button (**P** Fig. h). Then, to start the program, press and hold the increase intensity button (**+** Fig. i) until you feel your muscles contracting.


Fig. g

Fig. h

Fig. i

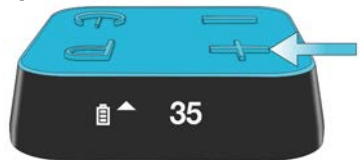Step 6:

- i) Continue increasing the intensity to a level that is comfortable, yet suitably challenging. This will vary from person to person.
- ii) Continue increasing the intensity throughout the session if possible. You will feel the muscle contractions getting stronger, but always reduce the intensity if you feel any discomfort.

NOTES: You can pause your toning session at any time. Simply press the on/off button briefly. The display will show a (II) symbol, indicating that the program is paused. Press this button again to resume the toning. If you want to stop your toning session before it is complete, press and hold the on/off button for at least two seconds. The controller will beep and switch off.

IMPORTANT: If you switch your controller off before a session is complete, that session and its maximum intensity will not be recorded. The three start-up screens will only display the values from the last completed session.

End of Session

- i) When the time reaches 00:00 the session is over and your SLENDERTONE COREFIT controller will stop automatically.
- ii) **IMPORTANT:** When you have finished toning, make sure you switch the controller off before removing the belt. Press and hold the on/off power button for two seconds and the controller will switch off.
- iii) Remove the belt and put the plastic covers back on the *black side* of the GelPads. Store the belt in a safe place until your next toning session.

Well done on completing your first training session!

PROGRAM DETAILS

Your SLENDERTONE COREFIT has 10 toning programs. The current Program number is displayed when you switch your controller on. Programs are classed as “Core Training” or “Core Movement”, as follows:

Core Training

| Program Name | Duration | Intensity Range |
|-----------------|----------|-----------------|
| 1. Initiation | 20 mins | 0 - 100 |
| 2. Beginner | 25 mins | |
| 3. Intermediate | 30 mins | |
| 4. Advanced | 30 mins | |
| 5. Expert | 30 mins | |
| 6. Pro | 30 mins | |
| 7. Ab Power | 30 mins | |

NOTE: Core Training programs can be used with or without other physical activity.

Core Movement

| Program Name | Duration / Crunches | Intensity Range |
|--------------------------|------------------------|-----------------|
| 8. Endurance | 40 mins | 0 - 100 |
| 9. Beginner crunch | 3 x 10 crunches | |
| 10. Advanced side crunch | 3 x 10 (side crunches) | |

PERFORMING CORE MOVEMENT PROGRAMS

Program 8

Program 8 can be performed while lightly jogging or doing a home exercise routine.

Step 1:

While wearing your belt, switch on the controller and select Program 8.

Step 2:

When you are ready to begin your exercise routine, press the increase intensity button to start the program. A countdown will appear on the display to allow you to get ready (Fig. j). After 3 seconds the toning will begin.

Step 3:

Begin your light cardio routine while slowly increasing the intensity until you find a strong, comfortable level that allows you to continue your workout while toning your abs.

Step 4:

As you become more comfortable with the intensity, increase it as far as possible to maximise the workout.

IMPORTANT: You should not ride a bicycle while using your SLENDERTONE COREFIT.

Fig. j

Programs 9 and 10 are performed while doing crunches in your own home.

Specific Ab Crunch precautions:

- Never do abdominal crunches if you suffer from neck or lumbar pain.
- Never exert yourself beyond your own comfort level. If in doubt, or if you have back pain consult your doctor before attempting these crunches.
- Programs 9 and 10 are designed for use ONLY with the abdominal crunches specified. No other form of exercise should be attempted with these programs.

Starting position (Fig. k):

Wearing your belt, lie on your back. Bend your knees to approx. 90°. Position your hands either side of your head. Your feet should be flat on the floor, shoulder width apart and your abdominal muscles fully relaxed. Switch your controller on and, choose Program 9 or Program 10 and then increase the intensity.

Fig. k

Program 9

Beginner Crunch (Fig. k):

1(a). When the stimulation starts, draw your navel in towards your spine and hold until stimulation stops. Relax until the next cycle and repeat.

Standard Crunch (Fig. l):

1(b). When the stimulation starts, slowly curl your shoulders forward, making sure that your lower back stays on the floor. Ensure you do not jerk your head forward. When raising your head, avoid neck strain by keeping a space approximately the size of your fist between your chin and your chest.

Fig. l

2. Hold each contraction for three seconds, then return to the starting position and relax fully. The stimulation will stop.

3. After a three second relaxation period, the stimulation starts again.

4. When you have done 10 crunches, the product begins a 20-second relaxation period. The controller will beep for the last three of the 20 seconds to indicate the start of a new set of crunches. Repeat step 1(a) or 1(b) and steps 2-3 depending on the type of crunch you are doing.

Note: After 3 sets of 10 reps the programme will end.

Program 10

Advanced Side Crunch (Fig. m) :

1. When the stimulation starts, slowly curl the left shoulder upwards and towards the midline/centre of your body to coincide with the contraction on the right side of your abdomen. Your shoulder should finish 3-4" from the ground. Always keep your lower back on the floor and never twist your upper body excessively.

Fig. m

2. Hold the contraction for two seconds, then return to the starting position and relax fully. The stimulation will stop.

3. Next, curl your right shoulder upwards and inwards when the stimulation begins on the left side of your abdomen. Again, hold for two seconds before returning to the relaxation point.

4. When you have done 10 crunches on either side, the product begins a 20-second relaxation period. The controller will beep for the last three of the 20 seconds to indicate the start of a new set of crunches. Repeat steps 1-3.

NOTE: After three sets of 20 reps (10 each side) the program will end.

ADDITIONAL FUNCTIONS

Mute Function on/off (Fig. n):

Press and hold the **P** button for two seconds to mute/unmute the controller.

NOTE: In the event of your product detecting an error, the controller will unmute to indicate the error. You must reset the mute manually if this occurs.

Keylock Function on/off (Fig. o):

Press and hold the Program and Intensity Up buttons for two seconds. Repeat this to unlock.

Error Messages:

In the unlikely event of your product developing a problem, the warning symbol and error code will appear on your controller's display (Fig. p). If this occurs, you should switch the controller off and then switch it back on again. It should now operate properly. If the problem persists, please contact

SLENDERTONE for further assistance at:

USA / Canada: 1-800-551-2443

International: +353 94 902 9936

Email: info@slendertone.com

Fig. n

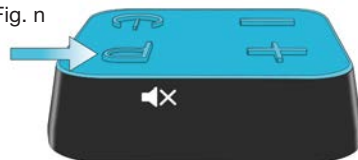
Fig. o


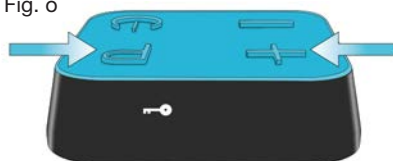
Fig. p

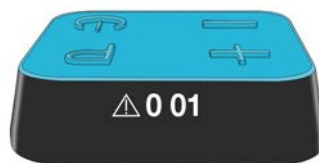GENERAL BATTERY ADVICE AND PRECAUTIONS

The controller contains 1 x 3.7 V Nominal Lithium Ion Polymer Battery.

Battery Charging & Discharging

- Your device is powered by a rechargeable battery. The battery can be charged and discharged hundreds of times, but it will eventually wear out. To ensure optimum battery performance you should use your device regularly.
- To conserve energy, disconnect the USB cable from the controller when the battery is fully charged.
- Leaving the battery in hot or cold places, such as in a closed car in summer or winter conditions, may reduce the capacity and lifetime of the battery. A device with a hot or cold battery may cease working temporarily, even when the battery is full charged. Battery performance is particularly limited in temperatures below freezing.
- Do not dispose of batteries in a fire as they may explode. Batteries may also explode if damaged. Dispose of batteries according to local regulations. Please recycle when possible. Do not dispose as household waste.

Charging Specifications

When charging the battery, you must use a charger with the following outputs:

Output Voltage: 5V DC nominal.

Output Current: 0.5A (500mA) minimum.

Output Connector: Micro-B USB Connector.

Expected Service Life:

Controller: 2 years

Belt: 2 years

GelPads: 20-30 Sessions

PRODUCT CARE

Caring For Your Belt

Your belt may be cleaned using a lightly dampened cloth, but you must first remove the controller and GelPads. Always follow the instructions on the label when washing the belt.

-  In order to protect the internal wiring from damage, do not hand wash or machine wash your belt.
-  Do not use bleach when washing the belt.
-  Do not dry clean your belt.
-  Do not tumble dry your belt. Dry the belt on a flat surface. Do not dry it over anything hot. (e.g. a radiator) as the belt contains plastic parts. Ensure the belt is completely dry before using it again.
-  The belt should not be ironed.

Belt Materials:

Outer material: 87% Nylon 13% Elastane; Hook & Loop: 100% Nylon; Inner Foam: 100% Polyurethane; Textured Panel: 70% Polyurethane, 30% Polyester.

Caring For Your Controller

- Your SLENDERTONE COREFIT controller should not be allowed to get wet or be left in excessive sunlight. It may be cleaned using a soft, lightly dampened cloth. Do not allow the interior of your controller to become wet. Do not use detergents, alcohol, spray aerosols or strong solvents on your controller. If your controller is damaged, you should not use it but you should contact the SLENDERTONE customer care helpline for advice. Repairs, service and modifications may not be carried out by anyone other than qualified service personnel authorised by SLENDERTONE.
- Access to the interior of the controller is not required for maintenance purposes. The battery is not accessible or replaceable.
- Do not try to open your controller casing or alter in product in any way that could cause harm to individuals. Tampering with the product would also invalidate the warranty.

Caring For Your GelPads

- The GelPads are for single person use only.
- The durability and effectiveness of the GelPads depends entirely on their use, storage and care. Certain skin types, the duration, number of sessions, intensity used and area of stimulation can also affect the life of the GelPads.
- Ensure your hands are clean before handling GelPads. You should avoid touching the black side of the GelPads with your fingers as much as possible when applying or removing the belt, as this can transfer oils and skin particles onto the GelPads' surface.
- After use, replace the liners on the **black side** of the GelPads and store your product in a cool, dry place until the next use.
- GelPads will need to be replaced periodically as the surface picks up skin debris and become dry over time.
- Do not use the GelPads if they are pierced or torn.

Rated Outputs – Voltage/Currents: Type 734

| Parameter | 500Ω | 1KΩ | 1K5Ω |
|-----------------------|-----------|-----------|-----------|
| Output RMSV | 7.35V | 13.1V | 15.5V |
| Output RMSA | 14.7mA | 13.1mA | 10.3mA |
| Output Frequency | 50-70Hz | 50-70Hz | 50-70Hz |
| DC Component: Approx. | 0 C | 0 C | 0 C |
| Positive Pulse Width: | 200-315µs | 200-315µs | 200-315µs |
| Negative Pulse Width: | 200-315µs | 200-315µs | 200-315µs |
| Interphase Interval: | 100µs | 100µs | 100µs |

TROUBLESHOOTING

| Problem | Possible Cause | Solution |
|--|---|---|
| Display doesn't come on; No signal from the controller; The battery symbol is flashing | The battery is low | Recharge the battery |
| The display is on but there is no stimulation | The session may be paused | Press the on/off button briefly |
| | Intensity not increased | Increase the intensity |
| ⚠ appears on the display* and/or the controller is beeping | Controller not properly connected to the belt | Ensure the controller and belt are properly connected |
| | The GelPads are on the wrong way around | Ensure the black surfaces of the GelPads are facing up |
| | The GelPads are worn | Replace your GelPads. Visit www.slendertone.com |
| | The GelPads are not covering the metal studs | Ensure each GelPad is covering the metal stud completely |
| | Poor GelPad contact with skin | Press the GelPads firmly against the skin. Refasten the belt tightly around your waist (see page 7) |
| | Build up of gel on the metal studs | Clean the metal studs of any debris with a dry cloth. Re-attach the GelPads |
| | Plastic covers are still on the GelPads | Remove plastic covers from both sides of the GelPads |
| Unpleasant feeling beneath the GelPads | The GelPads are worn | Replace your GelPads. Visit www.slendertone.com |
| | The GelPads are not covering the metal studs | Ensure each GelPad is covering the metal stud completely |
| | Build-up of gel on the metal studs | Clean the metal studs of any debris with a dry cloth. Re-attach the GelPads |
| | Too many consecutive sessions | Break for the day, allow your muscles and skin time to settle |
| The contractions are very weak even when the toning intensity is high | The GelPads are worn | Replace your GelPads. Visit www.slendertone.com |
| | The GelPads are not covering the metal studs | Ensure each GelPad is covering the metal stud completely |
| | Poor GelPad positioning | Reposition the GelPads according to your waist size (see page 6). Make sure belt is tightly fastened around your waist |
| ⚠ and a number appear on the display | Error message | Switch the controller off and on. The controller should work without any problems. If the error message persists, call our customer care helpline |

* Whenever the ⚠ symbol appears in conjunction with the left/right arrows (◀ / ▶), the left arrow indicates a problem with the left GelPad. If the right arrow appears, there is a problem with the right GelPad. If both left and right arrows appear, there is a problem with either the middle GelPad or all three GelPads.

TECHNICAL SPECIFICATIONS

Product type: 734

Intended use: Abdominal muscle stimulator

Waveform: Symmetrical bi-phasic square waveform when measured into a resistive load.

Accessories

Under no circumstances should anything other than SLENDERTONE accessories be used with your SLENDERTONE COREFIT product (Type 734). Any others may not be compatible with your product and could degrade the minimum safety levels. You can purchase all accessories from www.slendertone.com or by calling customer care.

SLENDERTONE COREFIT Belt

SLENDERTONE Abs GelPads:

- 1 large, square GelPad (Type 732)
- 2 small, oval GelPads (Type 733)

USB Cable: Type A to Micro B USB Charger

NOTE:

The signal gradually increases to a peak intensity level at the start of the contraction phase (i.e. ramp up) and gradually decreases to nothing at the start of the relaxation phase (i.e. ramp down). When the  symbol is flashing, the stimulus is significantly reduced.

Environmental Specifications:

Operating Range: Temperature: 41 to 104°F
Humidity: 15 to 93 % RH (non-condensing)
Atmospheric Pressure: 70 to 106kPa

Transport & Storage Range: Temperature: -13 to 158°F
Humidity: 10 to 93 % RH (non-condensing)
Atmospheric Pressure: 50 to 106kPa

The time required to reach optimal operating temperature range from transport and storage temperature range is up to 20 minutes, depending on the ambient temperature.

The Output Frequency indicates the number of pulses per second transmitted by the controller. This is measured in hertz, which is indicated by "Hz".

Output (RMSA) means the maximum output root mean square current for each channel.

Output (RMSV) means the maximum output root mean square voltage for each channel.

Attention: This product can deliver currents in excess of 2mA/cm² if used with incorrect electrodes.

Description of your unit's symbols:

There are a number of technical markings on your controller. These can be explained as follows:

 The product is manufactured for Bio-Medical Research Ltd, Parkmore Business Park West, Galway, Ireland.

 This symbol means "Attention, consult the accompanying documents".

 This symbol means type BF applied parts (electrodes).

 This symbol on your product is to indicate conformity to the requirements of the EMC Directive (2014/30/EU). The CE mark applied also indicates that this equipment complies with the requirements of the ROHS Directive 011/65/EU for the Restriction of Hazardous Substances in Electrical and Electronic Equipment.

SN stands for serial number. On the label on the back of the controller is the serial number specific to this product. The letter preceding the number indicates the year of manufacture, where 'X' denotes 2018, 'Y' denotes 2019 etc.

The belt's batch number is represented by the lot number label located underneath the wash label, by the number corresponding with the  symbol.

IP22 Ingress Protection Rating - this means it is protected against access to hazardous parts with a finger and against vertically falling water drops when tilted up to 15 degrees.

 At the end of the product lifecycle, do not throw this product or batteries into the normal household garbage, but bring it to a collection point for the recycling of electronic equipment. Some product materials can be re-used if you bring them to a recycling point. By re-using some parts or raw materials from used products you make an important contribution to the protection of the environment. Please contact your local authorities if you need more information about collection points in your area. Waste Electrical and Electronic Equipment can have potentially harmful effects on the environment. Incorrect disposal can cause harmful toxins to build up in the air, water and soil and can be harmful to human health.

 See page 16.

PRODUCT WARRANTY

Dear Customer,

Thank you for purchasing your SLENDERTONE product ("Product"). To be covered by this Limited Warranty, the Product must have been purchased directly from Slendertone Distribution, Inc. ("SDI"), or its authorized retailer in the United States. This Limited Warranty extends only to the original purchaser and cannot be transferred to anyone. Please read this carefully before using the Product.

Our Limited Warranty to you

SDI warrants to you, the original purchaser, that under normal use and service, the Product will be free from defects in material and workmanship for a period of two (2) years from the date of purchase ("Warranty Period"). If you discover a defect covered by this Limited Warranty during the Warranty Period, and return the Product to us, we will repair or replace the Product, at our option, using new or remanufactured components. There is no warranty or condition of any kind as to any defects discovered after the Warranty Period expires.

Certain Product failures not covered

This Limited Warranty covers defects in manufacture that arise from correct use of the Product per the product manual and instructions. It does not cover damage caused by abuse, misuse, accidents, modification or repair by any person other than SDI, moisture, extreme heat or cold, corrosive environments, high voltage surges from external sources or an unsuitable power supply. This Limited Warranty does not cover normal wear and tear on consumable items (e.g. pads, garments, etc.). This Limited Warranty does not apply to any battery provided with the Product, or if the Product has an altered or defaced serial number.

How to obtain service under this Limited Warranty

To obtain service under this Limited Warranty, please contact SDI's Customer CareLine at 1-800-551-2443 to obtain return instructions. You must submit proof of purchase in the form of a bona fide, dated bill of sale, receipt, or invoice (or copy thereof) evidencing that your request for service is made within the Warranty Period.

Except as may otherwise be required by legislation in your State, SDI will not be responsible for the cost of returning the Product or for any damage to or loss of the Product while it is in transit to us. We recommend that you use a carrier that will track your shipment and ensure the Product's safe delivery. SDI cannot accept any returns sent C.O.D.

THIS LIMITED WARRANTY IS THE ONLY WARRANTY FOR THE PRODUCT, AND THERE ARE NO OTHER EXPRESS WARRANTIES, ORAL OR WRITTEN, PROVIDED BY SDI. ANY IMPLIED WARRANTIES APPLICABLE TO THE PRODUCT, INCLUDING ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE WARRANTY PERIOD SET FORTH ABOVE.

Limitation of Remedies

YOUR EXCLUSIVE REMEDY, IN LIEU OF ALL INDIRECT, INCIDENTAL, CONSEQUENTIAL, EXEMPLARY, SPECIAL OR OTHER DAMAGES OF ANY KIND, INCLUDING FOR NEGLIGENCE, IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE PRODUCT, IF DEFECTIVE, UNDER THE TERMS AND CONDITIONS OF THIS LIMITED WARRANTY. SDI will bear no other expenses.

SDI reserves the right to refuse to repair or replace the Product if it is returned in an unsanitary condition. SDI reserves the right to substitute a product of equal or better quality if an identical product is not available at the time of replacement. This Limited Warranty is non-transferable.

Applicability of state law

Some states do not allow limitations on the duration of implied warranties, or the exclusion or limitation of incidental or consequential damages, and so the related limitations and exclusions stated above may not apply to you.

Governing law

This Limited Warranty is valid in the United States only and is not applicable in other countries. If you acquired the Product in the United States, the laws of the State of New York will apply to this Limited Warranty.

THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS, WHICH VARY FROM STATE TO STATE.

FC This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Caution: Changes / modifications not approved by the manufacturer could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

EMC INFORMATION

This medical device complies to the IEC 60601-1-2:2007 standard for Electromagnetic Compatibility.
See tables 1, 2, 4 and 6 for compliance details.

| Table 1 - Guidance and Manufacturer's Declaration - Electromagnetic Emissions | | |
|---|----------------|---|
| The SLENDERTONE COREFIT is intended for use in the electromagnetic environment specified below. The customer or the user of the SLENDERTONE COREFIT should ensure that it is used in such an environment. | | |
| Emissions Test | Compliance | Electromagnetic Environment - Guidance |
| RF emissions CISPR 11 | Group 2 | The SLENDERTONE COREFIT must emit electromagnetic energy in order to perform its intended function. |
| RF emissions CISPR 11 | Class B | The SLENDERTONE COREFIT is suitable for use in all establishments, including domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| Harmonic emissions IEC 61000-3-2 | Not applicable | |
| Voltage fluctuations/ flicker emissions IEC 61000-3-3 | Complies | |

| Table 6 - Recommended separation distances between portable and mobile RF communication equipment and the SLENDERTONE COREFIT. | | | |
|---|---|---------------------|---------------------|
| The SLENDERTONE COREFIT is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the SLENDERTONE COREFIT can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the SLENDERTONE COREFIT as recommended below, according to the maximum output power of the communications equipment. | | | |
| Rated maximum output power of transmitter (W) | Separation distance according to frequency of transmitter (m) | | |
| | 150 KHz to 80 MHz | 80 MHz to 800 MHz | 800 MHz to 2.5 GHz |
| | $d = 1.17 \sqrt{P}$ | $d = 1.17 \sqrt{P}$ | $d = 2.33 \sqrt{P}$ |
| 0.01 | 0.12 | 0.12 | 0.23 |
| 0.1 | 0.37 | 0.37 | 0.75 |
| 1 | 1.17 | 1.17 | 2.33 |
| 10 | 3.70 | 3.70 | 7.36 |
| 100 | 11.70 | 11.70 | 23.50 |
| For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (w) according to the transmitter manufacturer. | | | |
| NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. | | | |
| NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. | | | |

| Table 2 - Guidance and Manufacturer's Declaration - Electromagnetic Immunity - for all equipment and systems | | | |
|---|----------------------------|----------------------------|---|
| The SLENDERTONE COREFIT is intended for use in the electromagnetic environment specified below. The customer or the user of the SLENDERTONE COREFIT should ensure that it is used in such an environment. | | | |
| Immunity Test | IEC 60601 Test Level | Compliance Level | Electromagnetic Environment - Guidance |
| Electrostatic discharge (ESD) IEC 61000-4-2 | ± 6kV contact ± 8kV air | ± 6kV contact ± 8kV air | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%. |
| Power frequency (50/60Hz) magnetic field IEC 61000-4-8 | 3 A/m | 3 A/m | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |

| Table 4 - Guidance and Manufacturer's Declaration - Electromagnetic Immunity - for equipment and systems that are not life-supporting | | | |
|---|----------------------------|------------------|---|
| The SLENDERTONE COREFIT is intended for use in the electromagnetic environment specified below. The customer or the user of the SLENDERTONE COREFIT should ensure that it is used in such an environment. | | | |
| Immunity Test | IEC 60601 Test Level | Compliance Level | Electromagnetic Environment - Guidance |
| Radiated RF IEC 61000-4-3 | 3 V/m 80 MHz to 2.5 GHz | 3 V/m | <p>Portable and mobile RF communications equipment should be used no closer to any part of the SLENDERTONE COREFIT, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = [1.17] \sqrt{P}$ $d = [1.17] \sqrt{P} \dots 80\text{MHz to } 800 \text{ MHz}$ $d = [2.33] \sqrt{P} \dots 800 \text{ MHz to } 2.5\text{GHz}$ <p>Where P is the maximum output power rating of the transmitter in Watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a, should be less than the compliance level in each frequency range ^b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p> |

NOTE 1 : At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 : These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/ cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the SLENDERTONE COREFIT is used exceeds the applicable RF compliance level above, the SLENDERTONE COREFIT should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orientating or relocating the SLENDERTONE COREFIT

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

SLENDERTONE COREFIT is designed to support anticipated disturbance originating from electrostatic discharge, magnetic fields or radio frequency transmitters. However, the performance of the SLENDERTONE COREFIT device can still be affected by radio frequency fields originating from other sources some of which may not be visible, and the device could potentially be exposed to fields from these emitters without the user's awareness.

Whenever the user realizes an unexpected change in any of the stimulation parameters, it is advised to terminate the stimulation session immediately.

Equipment such as electrocautery or diathermy equipment, RFID readers, electronic article surveillance (EAS) systems or metal detectors that emit strong radio frequency signals can cause electrical interference which may affect device stimulation parameters. Ensure adequate separation between such emitters and the device during use.

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Copyright © 2018 Bio-Medical Research Ltd. All rights reserved.
SLENDERTONE is a registered trade mark of BMR Ltd.
Bio-Medical Research Ltd., Parkmore Business Park West, Galway, Ireland.